

experience

function

quality

style

technology



TILTLOCK® FOLDING GRAB RAILS brochure





Safe-Assist® TILTLOCK FOLDING GRAB RAILS for Healthcare Applications

The SAFE-ASSIST® TILTLOCK®, folding grab rail is designed for use in toilet suites, bath assist and shower recess enclosures and for use as well by persons having limited strength in areas where support or assistance is required.

The TILTLOCK®, folding grab rail with push button technology automatically locks when in the horizontal position and locks vertically as a grab rail for standing patients and makes wheelchair sliding transfers much easier, while providing comfortable safer sitting support.

Tested in accordance to comply with clause 17 of 1428.1-2009 for Design and Construction by Australian National Testing Laboratories Pty Ltd (ANTL). The TILTLOCK® was subjected to a weight loading of 1100N mass, directly upwards, 90 degrees to the right and 90 degrees to the left without any permanent deformation, distortion or loosening of the fastenings. In addition our standard TILTLOCK® options have been subjected to a down-force test of 280Kg. (Test report 4197-22.07.13). The Bariatric model was tested to a down-force test of 300Kg. (Test report 4418-23.10.2014).

For further information, refer to the TILTLOCK® options or Contact our Technical Adviser on the Priority Phone Number - 1800 472 272.

SAFE-ASSIST® TILTLOCK® OPTIONS

Specify left or right hand					
TILTLOCK® 850 mm GRAB RAIL			TILTLOCK® 850 mm GRAB RAIL WITH NURSE CALL HOOK		
 LH (Left Hand) PATENT NUMBER 2006100929	TL 850 SS LH	Satin Supreme™	 LH (Left Hand)	TL 850 SS LHH	Satin Supreme™
	TL 850 PS LH	Polish Supreme™		TL 850 PS LHH	Polish Supreme™
	TL 850 WT LH	White		TL 850 WT LHH	White
	TL 850 SS RH	Satin Supreme™		TL 850 SS RHH	Satin Supreme™
	TL 850 PS RH	Polish Supreme™		TL 850 PS RHH	Polish Supreme™
	TL 850 WT RH	White		TL 850 WT RHH	White
TILTLOCK® 850 mm GRAB RAIL WITH FIXED TOILET ROLL HOLDER			TILTLOCK® 850 mm GRAB RAIL WITH FLEXIBLE TOILET ROLL HOLDER		
 LH (Left Hand) Supplied with 19 mm stainless steel toilet roll holder secured to grab rail.	TL 850 SS LHS	Satin Supreme™	 LH (Left Hand) Supplied with Flexible ABS toilet roll holder fastened to grab rail.	TL 850 SS LHT	Satin Supreme™
	TL 850 PS LHS	Polish Supreme™		TL 850 PS LHT	Polish Supreme™
	TL 850 WT LHS	White		TL 850 WT LHT	White
	TL 850 SS RHS	Satin Supreme™		TL 850 SS RHT	Satin Supreme™
	TL 850 PS RHS	Polish Supreme™		TL 850 PS RHT	Polish Supreme™
	TL 850 WT RHS	White		TL 850 WT RHT	White
TILTLOCK® 850 mm BARIATRIC GRAB RAIL			TILTLOCK® 850 mm GRAB RAIL WITH NURSE CALL MOUNT BLOCK		
 RH (Right Hand) Reinforced for Bariatric requirements Max. load 300 kg	TL 850 SS LHB	Satin Supreme™	 LH (Left Hand) Fitted with 304 stainless steel nurse call mount box. Nurse call switch not supplied with TILTLOCK®.	TL 850 SS LHN	Satin Supreme™
	TL 850 PS LHB	Polish Supreme™		TL 850 PS LHN	Polish Supreme™
	TL 850 WT LHB	White		TL 850 WT LHN	White
	TL 850 SS RHB	Satin Supreme™		TL 850 SS RHN	Satin Supreme™
	TL 850 PS RHB	Polish Supreme™		TL 850 PS RHN	Polish Supreme™
	TL 850 WT RHB	White		TL 850 WT RHN	White

CUSTOMIZED DESIGNS AVAILABLE ON REQUEST




SAFE-ASSIST® TILTLOCK® FOLDING GRAB RAILS

The SAFE-ASSIST® TILTLOCK® 850 mm x 185 mm x 32 mm TILTLOCK® folding grab rail with safety lock mechanism, locks both horizontally and vertically. Fabricated with a chromed brass button release base housing, a solid brass concealed 4 fixed wall "T" mount bracket, no exposed nuts or bolts using 304 stainless steel tube. The TILTLOCK® grab rails are in compliance with the general requirements for applications in hospital, homes for elderly and nursing homes.

GRAB RAILS SHOULD BE MOUNTED SO THAT THE TOP OF THE RAIL IS 800-810 mm ABOVE THE FLOOR.




TOILET ASSIST - TILTLOCK®

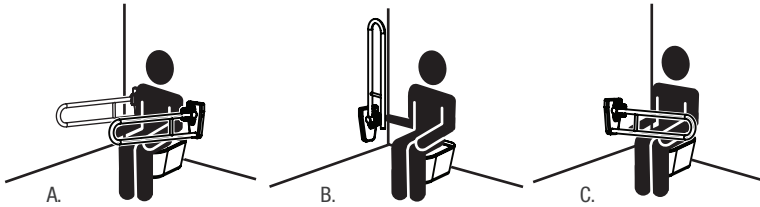


- A. Toilet Assist, Back wall, Twin Mount Tiltlock® rails, used when patient required the use of both horizontal locked support arms to rise and stand from the toilet pan.
- B. Toilet Assist, Side wall mounted Tiltlock® rail locked in the vertical position when patient needs to balance and feel comfortable when standing up from the toilet pan.
- C. Toilet Assist, Same side wall mounted Tiltlock® rail locked in the horizontal position, for patient whom requires a safe support to lower or pull forward with both hands to settle or rise from the toilet pan.

STANDING PATIENT - TILTLOCK®

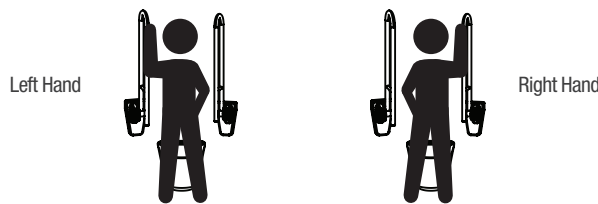


Toilet Assist, Back wall, Twin Mount Tiltlock® rails, locked in the vertical position for upright support when nature calls, to assist standing male patients with one hand on the rail arm for balance and one hand assisting the male member for directional flow into the toilet pan. Installation suites left or right handed patients. As shown.

BATH ASSIST - TILTLOCK®

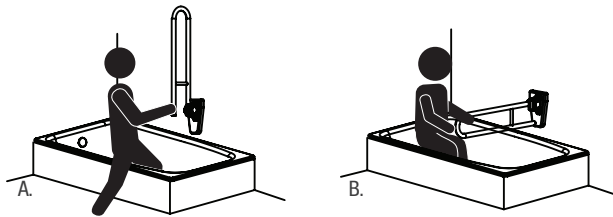


- A. Bath side assist wall mounted Tiltlock®, locked in the vertical position to give the bather assistance and security when entering the bath.
- B. Same wall mounted Tiltlock®, locked in the horizontal position, allowing the bather to grab the arm with both hands to lift and stand. Rail can then be re-locked in the vertical position to give assistance and security when leaving the bath.

SHOWER ASSIST - TILTLOCK®

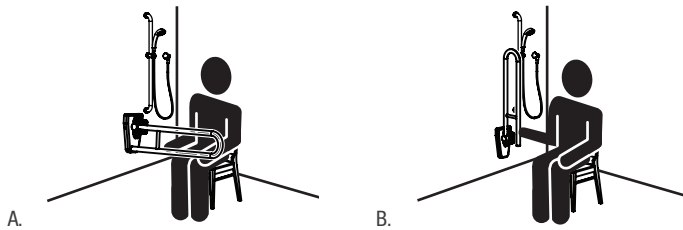

- A. Side wall shower Tiltlock®, rail locked in the horizontal position to assist standing or seated persons security when showering and who requires a safe support, using both hands to lower and settle onto a shower cubicle seat.
- B. Same side wall shower Tiltlock®, rail locked in the vertical position, for when a seated person needs a support to balance one self and feel comfortable when standing up from the shower cubicle seat.


SLIDING TRANSFER - TILTLOCK®

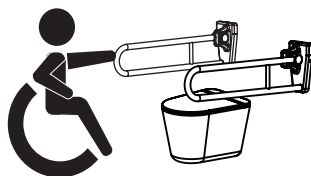

Toilet Assist, Back wall, Twin Mount Tiltlock®, locked in the horizontal position to support and aid wheelchair patients, making the transition from wheelchair to toilet pan much safer and more comfortable.

For those wheelchair users who leave the wheelchair in a frontal movement and slide transfer onto the toilet pan, the round rail ends of Tiltlock® arms makes it easier to pull forward with both hands from a wheelchair sitting position onto the pan, and likewise in reverse onto the wheelchair from the pan, when task is complete.

TILTLOCK® - TWIN MOUNT POSITION

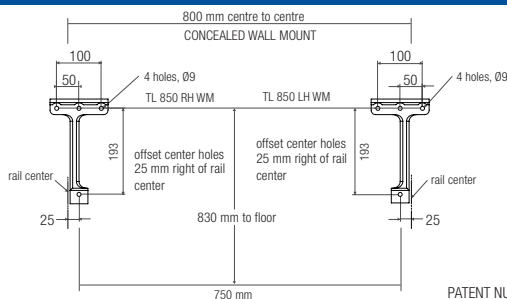

PATENT NUMBER: 2006100929

TILTLOCK® - FLOOR MOUNT UPSTAND



Stainless steel floor stand used with free standing toilets, Bariatric situations, and when wall mounting is not possible.
Stainless steel fixing screws supplied.



Protecting the world's most valuable liquid asset... water!™

OUR PROMISE

Con-Serv Corporation Australia Pty Ltd warrants all of its products (as below) with a Limited Lifetime Warranty on all mechanical parts from manufacturing defects in materials and workmanship under normal use in a domestic installation for as long as the original purchaser owns their home into which the particular product(s) was/were installed. A five year warranty (also on the terms below) will apply to commercial installations, irrespective of ownership.

Con-Serv will repair or replace defective items correctly installed as per the Con-Serv installation instructions. Con-Serv will not be liable for defects arising through normal wear and tear. Item(s) must be returned with proof of purchase to Con-Serv at 17 Glentanna Street, Kedron, Queensland 4031. We will attend to your warranty claim after we make appropriate investigations. Contact our Service Manager during normal office hours on 1300 467 322 or at our email address of warranty@con-serv.com.au. If a warranty claim is accepted, Con-Serv will bear the reasonable costs of the return of the defective item(s) to us. A claim for expenses associated with making a claim for a defective product must be in writing and submitted to us with copies of supporting documents. The benefits given under our warranty are in addition to other rights and remedies of the customer under a law in relation to goods and services to which this warranty relates. Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement or a refund for a major failure and compensation for any other reasonable foreseeable loss or damage. You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure. Visit the Con-Serv web site www.con-serv.com.au for the detailed warranty policy.



CON-SERV CORPORATION AUSTRALIA PTY. LTD.
17 Glentanna Street, Kedron · Qld. 4031 Australia
Phone: +61 (0)7 3630 5744 · Fax: +61 (0)7 3630 5755
Email: mail@con-serv.com.au · www.con-serv.com.au